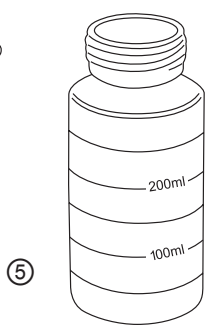
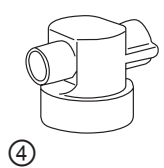
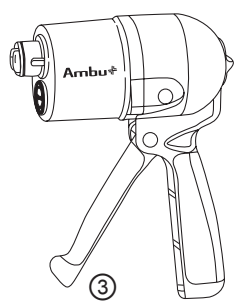
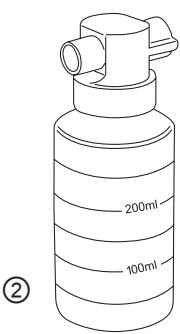
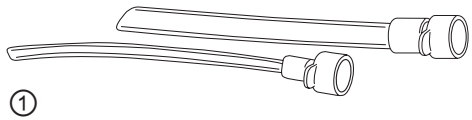
Instructions for use

Ambu[®] Res-Cue Pump[™]

Contents	Page
1. Warning and caution statements	4
2. Intended use.....	4
3. Specifications	5
4. Description	5
5. Operating instructions	5
6. Cleaning - disinfecting - sterilising	7
7. Test for correct function	7
8. Maintenance.....	8
9. Spare Parts.....	8
10. Accessories	8

Ambu® is registered trademark of Ambu A/S, Denmark.

Ambu A/S is certified according to ISO 9001 and ISO 13485.

Ambu® Res-Cue Pump™

Hand Held Portable Suction Unit

1. Warning and caution statements

WARNING

- Read instructions before use. The pump should only be used by persons with training in suction techniques.
- The use of the suction device may require that appropriate infectious disease precautions be taken. Precautions should be taken during use, cleaning of the device and disposal of the tubing and the container (as per your protocol).

CAUTION

- U.S. federal law restricts this device to sale by or on the order of a physician (USA and Canada only).
- Performance values given are achievable under test conditions. They may vary during actual use.
- Never immerse the pump unit in water or other liquids. If this happens, the pump should be replaced.
- If the pump is contaminated with liquids from the patient it should be replaced. Otherwise cross-contamination might occur.
- Following each cleaning and reassembly the pump should be tested for correct function (see section 7).
- Use only the supplied suction tubes and connectors.

2. Intended use

The **Ambu® Res-Cue Pump™** is intended to provide quick and effective pharyngeal and tracheal suction for multiple emergencies.

3. Specifications

The Ambu Res-Cue Pump is in conformity with the following standard: EN ISO 10079-2.

The Ambu Res-Cue Pump is in conformity with Council Directive 93/42/EEC concerning Medical Devices

Vacuum (max): -450 mmHg

Peak free airflow: > 20 l/min

Disposable container volume: 300 ml

External diameter of cap (suction port): 17 mm

External diameter of catheter connection: 13 mm and 6 mm

Stroke Adjustment: Turn stroke adjustment knob 90° counter-clockwise to generate 50% vacuum (of original pressure).

Operating environmental temperature: -20 °C to + 50 °C (-4 °F to 122 °F)

Storage environmental temperature: -40 °C to + 60 °C (-40 °F to 140 °F)

Dimensions: 185 x 70 x 168 mm

Weight: 260 g

4. Description

Ambu Res-Cue Pump is a light portable suction unit that can be powered by one hand and allows the other hand free for other important duties. This suction unit was designed to provide simple operation and maintenance. The adjustable stroke knob will provide different suction pressures.

5. Operating instructions

Select appropriate size tubing (according to the size of the patient) and attach it to the adapter (if not attached already). Attach the adapter with tubing to the cap of the container. The container should be attached to the pump.

CAUTION

It is very important that the container cap is attached all the way to the pump body.

Use of Ambu® Res-Cue Pump™

The Res-Cue Pump is operated by a simple squeezing action of the trigger. The adjustment stroke knob will enable it to produce various suction volumes (see below). Pull or squeeze the trigger toward the pump handle. This squeezing mechanism will generate the vacuum in the pump and suck the fluids into the container. Once suction is completed, remove tubing slowly from the patient. Remove filled container by unsnapping the container (and tubing) from the pump. Place container along with the tubing in the poly bag. Discard disposable tubing and container appropriately.

If required, place a label on the container for transportation to the laboratory.

Adjust the stroke

To adjust the stroke of the handle, turn the adjustment stroke knob. With the stroke adjustment knob seated at horizontal position, the stroke provides a maximum vacuum of -450 mmHg. With the stroke adjustment knob turned 90° counter-clockwise to vertical position, the pump will generate a maximum vacuum of -225 mmHg.

Overfill protection mechanism

The collection container is equipped with an overfill protection mechanism. Once the collection container is full, the overfill protection mechanism will be engaged. When the mechanism is engaged, it is not possible to squeeze the handle for further suctioning (extremely high resistance will be felt when trying to do so). When this happens, empty the container or use a backup container for further suctioning.

The overfill protection mechanism will ensure that the fluid will not enter the pump housing once the container is full.

CAUTION

- The pump shall not be shaken while pumping or during use.
- The pump should not be tilted for the angles more than 45 degree while pumping or during use.

If the pump handle becomes difficult to operate, a blockage may have occurred. Do not continue suctioning until the source of the blockage has been determined (remove the blockage from either the catheter, the adapter or the cap).

Adapters

One grey adapter tip for finger-tip connector and one adapter for funnel shape connector is included with the Res-Cue Pump. To use catheter with finger-tip connector and funnel shape connector for suction, attach larger end of the adapter tip to the canister cap and catheter to the smaller end of the adapter tip. Standard suction catheter tubing with finger-tip connector and funnel shape connector is not included. Follow your local protocol for all suctioning practice.

Note: If the pump is turned upside down, the overflow protection mechanism will be engaged making it impossible to suction. If sudden resistance is felt during suctioning and at the same time container is not full then the overflow protection mechanism might have become engaged. If the overflow mechanism is activated and the valve blockage occurred, the container should be removed from the pump unit to deactivate the overflow mechanism and release the stuck valve. If it is still difficult to operate the pump handle and resistance is felt, check for the blockage as described earlier.

6. Cleaning - disinfecting - sterilising

Disassembly

Remove the container by unsnapping the cap with attached container ② and tubing ① from the pump ③.

Disinfect the pump and handle (by scrubbing with warm soapy water, mild detergent, bleach solution etc.) before placing into storage.

Assembly

To assemble the Res-Cue Pump after cleaning screw a new container ⑤ into the cap ④ and push the cap firmly onto the connector of the pump ③ until it snaps in.

Select appropriate size tubing ① (according to the size of the patient) and select it to the adapter (if not attached already). Attach the adapter with tubing to the cap of the container ②.

7. Test for correct function

Check the handle's condition and the pump function before returning it to service. Mount a container, occlude the suction port by thumb or palm and squeeze the pump handle to check for the vacuum.

8. Maintenance

The Ambu Res-Cue Pump requires no special maintenance or storage conditions.

9. Spare Parts

276 000 010 Disposable container (Includes one disposable container, cap, disposal bag, 2 suction catheters 13 and 6 mm)

10. Accessories

276 000 011 Carry case (Holds pump, 2 disposable containers and 4 suction catheters)

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Ambu B.V.