The Ambu® ResKey is a protective single-use mask with a filter for mouth-to-mouth or mouth-to-nose ventilation. It provides a physical barrier and eliminates direct contact between the rescuer and the victim. The Ambu ResKey complies with the German standard DIN 13154.

Instructions for use

- Take the Ambu ResKey out of the package, unfold it (1) and remove the inlay paper.
- Place it over the victim's face and place the elastic bands behind the victim's ears for fixation (2).
- Place the Filter over the victim's mouth for mouth-to-mouth ventilation (3a) or over the victim's nose for mouth-to-nose ventilation (3b).
- Open the victim's airway using head tilt and chin lift then start ventilation (4).
- If there is no chest rise during ventilation, check adequate head tilt and chin lift.
Failure to do so may result in insufficient ventilation.
- Continue ventilation/CPR until the victim starts breathing normally or qualified help arrives and relieves you.

Perform resuscitation according to the present resuscitation guidelines. Further detail on resuscitation can be found on the website of the www.erc.edu (Europe) or the www.americanheart.org (US).

For single use only. Re-use can result in cross infection.

If you cannot ventilate the victim or you do not have your Ambu ResKey start ventilation without the Ambu Reskey without delay.

Ambu® is a registered trademark of Ambu A/S, Denmark.

Ambu ResKey is in conformity with Council Directive 93/42/EEC concerning Medical Devices.

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Ambu® är ett registrerat varumärke som tillhör Ambu A/S, Danmark.